POWERbreathe
Shaker Medic Plus

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İthalatçı Firma

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Dezenfektan İçeriği	Örnek	Konsantrasyon	Yorum
Chlorine dioxide generator	Tristel one day	20ml in 1 litre water	Safety data sheet of water and further information available from www.tristel.com
Sodium hypochlorite	Milton	1000 ppm	Ensure thorough rinsing, (NaOCL) corrosion of the metal parts 4 as will occur if exposed to chlorine for long periods.
Sodium dichloroisocyanurate	POWERbreathe Cleansing Tablets	1000 ppm	Follow instructions provided by the manufacturer

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Referanslar

Chemical disinfection in hospitals – PHLS, Sterilization, disinfection and cleaning of medical device equipment (MAC Manual) – MHRA

Not:

1. This method of cleaning has been independently tested and validated to ensure that if followed the Shaker Medic Plus will be sufficiently clean for re-use.
2. Lancerzyme® is a registered trade mark of Lancer UK Ltd (www.lancer.co.uk) other options may be available locally in other countries
3. The amount of time the device is left in the solution will depend on the disinfectant chosen.
4. The ball in the Shaker Medic Plus is nickel plated carbon steel; in tests some balls have shown signs of discolouration and scratches after prolonged submersion in some cleaning solutions. However, the functionality of the product has not been affected.
5. It is the responsibility of the user to ensure all sterility verifications The Shaker Medic Plus cannot be sterilized with ethylene oxide or hot air.

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